Corsinel®

by Tytex

Ostomy pouch fixation and support tube | تثبيت حيب الفغر ولفافة الدعم الأنبوبية | Držáč za fixiranje i podupiranje vrećice za stomu | Fixace stomického sáčku a podpurná trubička | Fikserings- og støtte bandage til stomipose | Bandage zur Fixierung und Stützung eines Stomabeutel | Fijación de bolsa de ostomía y tubo de soporte | Stoomikotti fikseeriv ja toetav tuub | Avannepussin tukituubi | Tube de fixation et de soutien de la poche de stomie | Sztómazsák és haskötő | Festing fyrir stómapoka og kviðstuðningsteygju | Fissaggio della sacca per stomia e del tubo di supporto | オストミー用パウチの固定とサポートチューブ | 장루 주머니 고정 및 지지 튜브 Fixatie- en ondersteuningsbandage voor stomazak | Støttebandage til fiksering af stomipose og støtte bandage | Mocowanie worka stomijnego i pasa brzuszneho | Fixação de bolsa de ostomia e tubo de suporte | Трубка для фиксации и поддержки калоприемника | Fixácia stomického vrečka a oporná tuba | Fastsättning av stomipåse och stödtub | 造口袋的固定和支撑针织套

Instructions for Use | لام عتس ال ا تمامي لع ت | Uputstvo za upotrebu | Návod k použití | Brugsanvisning | Gebrauchsanweisung | Instrucciones de uso | Kasutusjuhend | Käyttöohje | Instructions d'utilisation | Használati utasítás | Leiðbeiningar um notkun | Istruzioni per l'uso | 使用说明 | 사용 지침 | Gebruiksaanwijzing | Bruksanvisning | Instrukcja użytkowania | Instruções de uso | Инструкции по применению | Návod na použitie | Bruksanvisning | 使用说明

3216 StomaSafe Plus | 5000 StomaSafe Classic

Intended purpose

Fixation, concealment and support of an ostomy pouch or another medical device

Intended users

Users with a colostomy, an ileostomy, or an urostomy

Indications

For post-surgical use after construction of a colostomy, an ileostomy or a urostomy. For use with both 1- and 2-piece systems

 Use the device according to the indications to prevent injury or damage

Contraindications

Relative contraindications

- Skin diseases
- Contact dermatitis

Warnings and precautions

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to prevent injury or damage
- The device may influence the stoma drainage if it is too small or fitted too tight
- Remove the device if any unexpected discomfort, stoma or skin irritation occurs
- By urgent need for removal of the device, open with scissors

Performance characteristics

General Performance Characteristics

	3216 StomaSafe Plus	5000 StomaSafe Classic
 4-way stretch: Increased freedom of movement and supportive fit	X	X
 Breathable: Air permeable and moisture absorbing	X	X
 Seamless: The device is knitted in one piece without seams	X	

Specific performance characteristics

- Double layered device with a hole for the ostomy pouch in the inner layer, preventing contact between skin and pouch
- The outer layer retains the ostomy pouch safely, preventing the pouch from moving
- Contributes to reducing the risk of leakage from the pouch
- Contributes to a sense of control over body function and appearance
- Helps reduce stoma noise

Material composition

 1

PRODUCT	MAIN FABRIC COMPOSITION
3216 StomaSafe Plus	91% PA, 9% EL
5000 StomaSafe Classic	49% CO, 24% PA, 18% EL, 9% PES

Raw materials and components used in the device are tested for harmful substances according to STANDARD100 by OEKO-TEX®, Annex 4, product class II.

The device and packaging are:

- Biologically safe, non-toxic; chlorine and dioxin free, and do not contain human or animal tissue or extracts, prohibited azo-dye pigments, medical substances or phthalates
- Free from and unexposed to natural rubber latex or natural rubber derivatives
- Free from heavy metals or other harmful contamination and do not pose any harmful human impact associated with its intended end-use
- In compliance with REACH
- No chemicals added during treatment/finishing of the device. The device may contain softening and antistatic agents up to 1.5 %

Care instructions

	3216 StomaSafe Plus	5000 StomaSafe Classic
 40°C or 104°F	X	
 60°C or 140°F		X
	X	X
Wash with similar colours	X	
Wash in a laundry bag	X	X
Gentle tumble drying	X	X
Use of chlorine based bleach or chlorine based disinfection will decrease durability	X	X

Measuring size

- Measure yourself in a standing position and at the widest abdominal circumference covered by the device.

EN

 Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size for an extra tight-fitting device and the larger size if you prefer the device looser. Ensure that the device feels comfortable to wear when sitting and standing.

Recommended size chart

- Use the size guide to find the right size based on your measurement.  2

Instructions for use

The device in its primary packaging is prepared for direct clinical use.

Fitting 3216 StomaSafe Plus

STEP	INSTRUCTION
1	 The text on the inner layer must be located on the inside at the top. Pull up StomaSafe Plus like underwear and make sure that the hole on the inner layer is positioned in front of the ostomy pouch.
2	 Fold up the outer layer, until the hole on the inner layer is fully visible and pull the ostomy pouch carefully through the hole.
3	 Fold up the ostomy pouch and place the bottom part in the small pocket at the bottom of the outer layer.
4	 Fold down the outer layer so that it covers the ostomy pouch.

Fitting 5000 StomaSafe Classic

STEP	INSTRUCTION
1	 Pull up StomaSafe Classic like underwear, with the large opening between the layers facing upwards.
2	 Pull the pouch through one of the two holes in the inner layer. Choose the hole that fits your pouch-location best and provides you the best comfort and support.
3	 Place the ostomy pouch in the inner pocket.
4	 When using a drainable pouch, pull the drainage valve through one of the openings at the bottom. Choose the opening that fits the placement of the drainage valve best.

Additional instructions

Single patient multiple use

Keep away from direct sunlight

Keep dry

May be disposed with regular household waste

Reporting obligation

If you have any complaints, please contact your supplier directly.

In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Lifetime of the device in use

The lifetime of the medical device is determined by natural wear and tear, even when treated correctly and as recommended. In case of signs of deterioration affecting performance and comfort, the device should be removed.

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Corsinel is a registered trademark of Tytex A/S © 2022.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast
Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • +45 9660 4200 • www.tytex.com