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Discreet and comfortable hip protection

SAFEHIP® AirX® DISCREET

Hip protector · Hüftprotektor
Protecteur de hanches · Heupbeschermer
Hoftebeskytter · Höftskyddsbyxor · Lonkkasuoja
Protector de cadera · Protezione dell'anca · Csípővédő

SAFEHIP®
THE HIP PROTECTOR

AirX®
100% BREATHABLE
100% TEXTILE

SAFEHIP®
THE HIP PROTECTOR

The protective shields in SAFEHIP® AirX® Discreet are based on the special horsehoe-shape, which not only reduces the energy of the impact of the fall, but also diverts it into the surrounding soft tissue.

The SAFEHIP® AirX® Discreet hip protectors have a unique and thin design which makes them discreet to wear – thereby providing ultimate discretion as well as protection. The pant and the 100% textile shields are moisture transporting and breathable which makes them very comfortable to wear.

Protector
100% AirX® soft textile

Seamless Briefs
94% Polyamide
6% Elastane

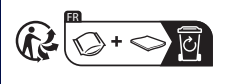
Care handling

SAFEHIP® can be washed at 60°C/140°F and tumble dried. Wash with similar colours and in a washing bag. Use of chlorine based bleach or chlorine based disinfection will decrease durability and function.

Please keep this instruction for further reference.

Tytex Customer Service:
E-mail: customerservice@tytex.com
International: +45 9660 4200; +49 2262 762914
Americas: +1 401-762-4100

SAFEHIP®	Cm	Inch	Find your size
S	75-95	30-38	
M	90-105	36-42	
L	100-115	40-46	
XL	110-130	44-52	
XXL	125-150	50-60	

Sizes overlap. To find the correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the thighs are slim, and the larger size if thighs are wide.

BIOMECHANICALLY
TESTED

TYTEX®
MAKING HEALTHCARE MORE HUMAN

Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
Made in Slovakia by: Tytex A/S · Marsvej 6 · DK-7430 Ikast · info@tytex.dk · www.tytex.com

User Manual

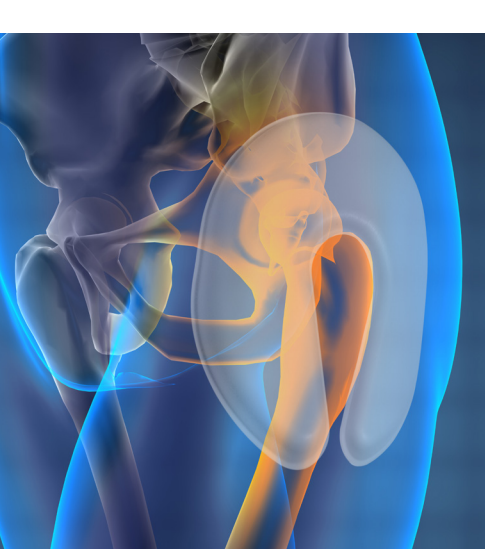
Diagram 1 / Figur 1 / Kuvan 1 / Abb. 1 / Figure 1 / Diagrama 1 / az 1. ábrának / Diagrama 1 / Rysunku 1

EN

Indication

Medical device for prevention of hip fracture in the event of a fall.

Features

SAFEHIP® AirX® Discreet is a hip protector with integrated, protective shields. The horseshoe shaped shields protect the critical hip area, exposed to trauma in the event of a fall on the side. The shields both absorb and disperse the energy away from the critical area, providing protection against hip fractures. The hip protector is developed so that the shields will sit in the right position when applying the pant, and the slim design of the shields offers discretion. The hip protector is made of a soft, elastic and breathable fabric that wicks moist away from the body, making the hip protector very comfortable to wear.

SAFEHIP® AirX® Discreet is available in a female and a male version in black or white, and it can be used as underwear or over underwear day and night.

Instructions for use

- It is recommended to sit down when applying the pant.
- Ensure that the text "FRONT" on the inside of the waistband is placed on your belly.
- Pull the pant up above your knees.
- Stand up and pull up the pant entirely so that it sits comfortably.
- Make sure that both shields are correctly centered over the critical area, the hip. The Safehir shields provide a protective ring around the bony prominence as shown in diagram 1.

Precautions

No hip protector can be 100% effective in preventing injury from falls, and wearing SAFEHIP® does not guarantee protection from injury. It is important always to ensure correct shield positioning, every time SAFEHIP® is applied. SAFEHIP® has been designed to resist multiple falls. We recommend that you check for any irregularities during the lifetime of the product and after each fall. If any damage occurs, the product must not be worn. Any persistent indentation is to be considered damage.

If you are malnourished, very slim or have very low subcutaneous fat deposits the bony prominence of the hip may protrude in the center of the horseshoe shield. In such case, we recommend that you use a SAFEHIP® model with thicker shields, e.g. SAFEHIP® AirX® or SAFEHIP® Classic.

SAFEHIP® is only to be used for its intended use of hip protection - it is not to be used in conjunction with other medical devices without consulting a doctor first.

If any unexpected discomfort or skin irritation occurs, please consult your doctor or nurse.

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