

Corsinel®
by Tytex

SUPPORT BELT



HIGHLY BREATHABLE



4-WAY STRETCH



SEAMLESS KNITTING



FULLY OPENABLE



LATEX FREE

OSTOMY AND HERNIA SUPPORT BELT



ABOUT SUPPORT BELT

- ✕ OFFERS DISCRETION AND RELIEF FROM DRAGGING SENSATION
- ✕ SOFT, SEAMLESS AND BREATHABLE FOR OPTIMUM COMFORT
- ✕ MAXIMUM SUPPORT FOR OPTIMAL HERNIA MANAGEMENT
- ✕ POCKET FOR EASY APPLICATION



STANDARD 100

6234-7001 DTI

EUROPEAN-BASED PRODUCTION
FOR OPTIMAL QUALITY,
SUPPLY CHAIN EFFICIENCY
AND FLEXIBILITY



Corsinel®

by Tytex



CORSINEL BELT
ARTICLE NUMBER: 3519


AVAILABLE COLOUR



COMPOSITION

54 % PA, 20 % CO,
13 % EA, 5 % PP


CARE HANDLING

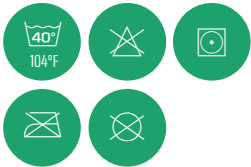

- Close hook-and-loop before washing
- Wash with similar colours and in a washing bag
- Use of chlorine based bleach or chlorine based disinfection will decrease durability and function



AVAILABLE IN SIZES

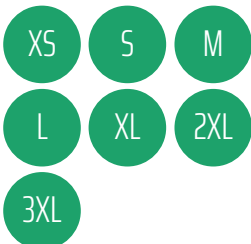

PRODUCT RANGE

Belt 20 cm, art. no. 3519

NOTE

For more support go down one size and for less support go up one size.

IMPORTANT: Always measure yourself to find the right size.

ABOUT THE PRODUCTS

The Corsinel Belt is designed to fixated, support as well as retain stomas and abdominal hernias. The belts offers maximum support to alleviate the uncomfortable feeling of a dragging hernia, making daily and strenuous activities easier to perform. Use of the belt also enhances a smooth appearance, providing discretion and encouraging participation in social life.

The belts have 4-way-stretch, which provides perfect adaption to body contour. Skin friendly material ensures no allergic reaction, redness or skin irritation in general. Seamless knitting offers optimum wearer comfort. The belts are designed for all day use.

FEATURES

- Maximum support for optimum hernia management
- 100% seamless knitting prevents skin irritation for highest user comfort
- Silicone anti-roll pattern
- 4-way stretch for optimum support and freedom of movement
- Air and moisture permeable material
- Latex-free material and 100% seamless knitting prevents skin irritation for highest user comfort
- Tested for harmful substances according to Oeko-Tex standard 100
- Easy-to-apply pocket and wide hook-and-loop for adjustment of support level
- Low cut back

INDICATION

For postsurgical use after construction of a colostomy, an ileostomy or an urostomy. For management of parastomal- or other abdominal hernias. For use with both 1- and 2-piece systems.

PRECAUTION

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to avoid injury or damage
- Make sure the device is not compromising stoma and/or lung functions
- If your hernia is irreducible (cannot be returned by manipulation), seek the advice and guidance of your ostomy nurse before using the device
- If any unexpected discomfort, stoma or skin irritation occurs, remove the device
- By urgent need for removal of the device, open with scissors
- Sizes overlap; if abdominal measurement lies between two sizes, choose the smaller size if you prefer extra tight-fitting garments, and the larger size if you prefer the garment less tight.
- Ensure that the garment feels comfortable to wear when sitting and standing.
- The device may contain softening and antistatic agents up to 1.5%

RECOMMENDED SIZES

15 - 20 CM

SIZE	CM	IN
XS	74 - 82	29 - 32
S	78 - 90	31 - 35
M	86 - 98	34 - 39
L	94 - 106	37 - 42
XL	102 - 116	40 - 46
2XL	112 - 130	44 - 51
3XL	126 - 148	50 - 58



Measurement at the widest abdominal circumference

WWW.CORSINEL.COM

CE TYTEX A/S HAS BEEN CERTIFIED ACCORDING TO ISO 9001, ISO 14001 AND ISO 13485