# safehip®

## by Tytex

Hip protection trousers or belt

Instructions for Use | تعليمات الاستعمال | Návod k použití | Brugsanvisning | Gebrauchsanweisung | Instrucciones de uso | Käyttöohje | Instructions d'utilisation | Használati utasítás | Istruzioni per l'uso | 使 説明 | 사용 지침 | Gebruiksaanwijzing | Bruksanvisning | Instruções de uso | Návod na použitie | Bruksanvisning | 使用说明

**EN** The device in primary packaging is prepared for direct clinical use. **AR** يكي نيل كمال م ادخستال ل ج ا نم قيس اس اتاوب ع يف زاو جلا انه زي هجت م متي رش اسجل **CS** Zdravotnický prostředek v primárním balení je připraven k přímému klinickému použití. **DA** Produkt i emballage forberedt til direkte medicinsk brug. **DE** Produkte in einer Primärverpackung sind gebrauchsfertig und können direkt für eine klinische Nutzung verwendet werden. **ES** El producto sanitario del envase está preparado para su uso clínico directo. **FI** Kuluttajapakkausessa oleva tuote on valmis suoraan kliniseen käyttöön. **FR** Dans son emballage primaire, le dispositif est préparé pour une utilisation clinique directe. **HU** Az elsődleges csomagolású eszköz közvetlen klinikai felhasználásra készül. **IT** Il presidio presente nella confezione è adatto all'uso clinico diretto. **JP** 本製品は直接包装されており、臨床にもご使用いただけます。 **KO** 기본 포장재의 기기는 직접 임상적 사용을 위해 준비되어 있습니다. **NL** Het hulpmiddel in primaire verpakking wordt voorbereid voor direct klinisch gebruik. **NO** Enheten i primærpakningen er klargjort for direkte klinisk bruk. **PT** O dispositivo na embalagem primária está preparado para utilização clínica direta. **SK** Pomůcka v spotřebitelskom obale je připravená na priame klinické použitie. **SV** Produkten i primärpackningen är förberedd för direkt klinisk användning. **CN** 初级包装中的本品可直接用于临床。

3265 SAFEHIP Classic Unisex, 3365 SAFEHIP AirX Unisex, 3366 SAFEHIP AirX Female, 3367 SAFEHIP AirX Male, 3368 SAFEHIP AirX Open, 3604 SAFEHIP AirX Discreet Female, 3612 SAFEHIP AirX Discreet Male, 8190 SAFEHIP Active, 0761 Classic Shield, 8110 AirX Shield

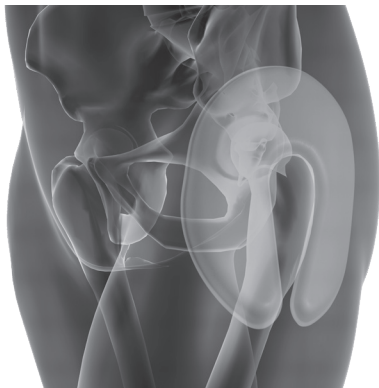
Figure 1, 1 شکل | Obrázek 1, Figur 1, Abbildung 1, Figura 1, Kuvio 1, Figure 1, 1. ábra, Figura 1, 図 1, 그림 1, Figur 1, Figure 1, Figura 1, obrázku 1, Figur 1, 图 1

**Intended purpose**

SAFEHIP®: Device to be worn by elderly people as part of a preventive garment for hip protection.

Shields: External device used to reduce the risk of hip fractures.

**Intended users**

Elderly people who have fallen or are at risk of falling.

**Indications**

- SAFEHIP (3265, 3365, 3366, 3367, 3368, 3604, 3612, 8190): For minimising the risk of hip fractures, in case of a fall.
- Accessories (0761, 8110): Accessory for use in hip protectors.

**Contraindications**

Absolute contraindications:

- Do not use AirX Discreet (3604, 3612) if you are malnourished, very slim or have very low subcutaneous fat deposits

Relative contraindications:

- Skin diseases
- Contact dermatitis

**Clinical benefits**

Reduces risk of hip fracture

**⚠ Warnings and Precautions**

- Use the device according to indications to prevent injury or damage

**SAFEHIP Classic - SAFEHIP AirX - SAFEHIP AirX Discreet - SAFEHIP Active**

- SAFEHIP® provides protection against hip fractures when both SAFEHIP® shields are placed correctly in both pockets of the SAFEHIP® briefs/belt.
- No hip protector can be 100% effective in preventing injury from all falls and wearing SAFEHIP® does not guarantee protection from injury.
- To avoid reduced performance, it is important that both shields are correctly centred over the greater trochanter.
- We recommend that you check for any irregularities during the lifetime of the product and after each fall.
- If any damage occurs the product must not be worn. Any tear, scratch or persistent indentation is to be considered damage.
- When using SAFEHIP® it is important to check the area of the body in direct contact with the product for signs of skin deterioration. If you see any examples of skin deterioration, please seek medical advice.
- SAFEHIP® is only to be used for its intended use of hip protection and incontinence pad fixation - it is not to be used in conjunction with other medical devices without first consulting a doctor.
- Do not use bleach as this will damage the SAFEHIP® over time. Bleach powder should not be used under any circumstances.
- In connection with industrial laundering SAFEHIP® should never be subjected to excessive pressure during the washing or drying process as pressure combined with heat may deform the shields
- Removable shields should be placed in the briefs before using. The shields must be placed in the pockets, with the legs of the horseshoe contour pointing down.
- If any unexpected discomfort or skin irritation occurs, remove the SAFEHIP®
- Follow the Instructions for Use
- SAFEHIP® only provides hip protection when worn

Additional for SAFEHIP Classic	Additional for SAFEHIP AirX	Additional for SAFEHIP AirX Discreet	Additional for SAFEHIP Active
<ul style="list-style-type: none"> <li>• SAFEHIP® can be washed at up to maximum. 60° C / 140 °F. Washing at a higher temperature will deform and/or shrink the shields, and if this happens, use must be discontinued.</li> <li>• Safehip® shields should only be used in combination with Safehip® briefs.</li> </ul>	<ul style="list-style-type: none"> <li>• Safehip® shields should only be used in combination with Safehip® briefs.</li> </ul>	<ul style="list-style-type: none"> <li>• If you are malnourished, very slim or have very low subcutaneous fat deposits the bony prominence of the hip may protrude in the center of the horseshoe shield. In such case, we recommend that you use a SAFEHIP® model with thicker shields, e.g., SAFEHIP® AirX® or SAFEHIP® Classic.</li> </ul>	<p>SAFEHIP® Active is not intended to be used in sports activities. Activities where the energy in falls may be considerably greater than that of daily life activities, the protection offered by the belt may be reduced and in worst case insufficient to reduce the fracture risk.</p>

**Performance Characteristics**

Attenuate impact force on hip/femoral neck

Disperse impact forces

Absorb impact forces

Can resist multiple hits

**Recommended size chart**

Size recommendations are to be used as a guide only.

Sizes overlap. To find the user's correct size, measure the hips at the widest point. If the hip measurement lies between two sizes, choose the smaller size if the patient has slim thighs, and the larger size if thighs are wide or an incontinence pad will be worn.

**Instructions for use**

For: 3265, 3365, 3366, 3368, 3604, 3612

1. It is recommended to sit down when applying the pant.
2. Ensure that the text "FRONT" on the inside of the waistband is placed on your belly.
3. Pull the pant up above your knees.
4. Stand up and pull up the pant entirely so that it sits comfortably.
5. Make sure that both shields are correctly centered over the critical area, the hip. The SAFEHIP® shields provide a protective ring around the bony prominence as shown in Figure 1.

For 8190:


1. It is easiest to apply the belt while standing.
2. Make sure that the belt is turned correctly. It is very important that the open end of the horseshoe shaped shields is facing downward towards the feet.
3. Position SAFEHIP® Active so that both shields are correctly centered over the critical area, the hip. The SAFEHIP® shields provide a protective ring around the bony prominence as shown in Figure 1.
4. Close the hook-G-loop fastening in the front and adjust for comfort and fit. The belt should be adjusted so that it does not shift or turn around the body. Do not tighten excessively so it impairs your comfort.
5. Please check that the SAFEHIP® Active and especially the shields are positioned as shown in Figure 1.
6. At regular intervals during daily activities, it is recommended to check that SAFEHIP® Active is positioned correctly according to these instructions.

If in doubt about appliance or fitting, please consult a doctor or nurse to guide you in fitting the belt correctly.

Only the textile part of the devices is in direct contact, with the human body. Shields are integrated into the garments.

**Recommended Care Instructions**


For 3265, 3604, 3612:

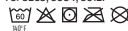For 8190:

For 3365, 3366, 3367, 3368:

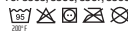- Wash with similar colours
- Wash in a laundry bag
- Gentle tumble drying
- Use of chlorine based products may decrease durability and function

Raw materials and components used in the device are tested for harmful substances according to STANDARD 100 by OEKO-TEX®, product class II, and relevant Biological Evaluation (ISO 10993). The device and packaging comply with REACH.

⚠ The device may contain softening and antistatic agents up to 1.5%.

**Reporting obligation**

If you have any complaints, please contact your supplier directly.

In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

**Lifetime of the device in use**

🕒 The lifetime of the medical device is determined by natural wear and tear, even when treated correctly and as recommended. In case of signs of deterioration affecting performance and comfort, the device should be disposed of.

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Safehip is a registered trademark of Tytex A/S © 2024.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast
Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • [www.tytex.com](http://www.tytex.com)