carefix®

Post-Surgical Bras | حمالات الصدر المخصصة للارتداء بعد العمليات الجراحية | Pooperační podprsenky
Post-operative bh'er | Post-OP-BH | Μετεχειρητρικοί στηθόδεσμοι | Sujetadores postquirúrgicos
Rintaliivit leikkauksen jälkeiseen käyttöön | Soutiens-gorge post-chirurgicaux | Postoperativni grudnjaci
Műtét utáni melltartók | Reggisenpost-chirurgici | 術後用ブラジャー | 수술후보정용 브라 | Postoperatieve beha's
Post-kirurgiske BH | Biustonosze pooperacyjjne | Sutiās pós-operatórios | Послеоперационные
бюстгальтеры | Pooperačné podprsenky | Modrčki za uporabo po kirurškem posegu | Post-hirurški
grudnjaci | Postoperativa brösthållare | 术后文胸

Instructions for Use | تعليمات الاستعمال | Návod k použití | Bruksanvisning | Gebrauchsanweisung | Οδηγίες χρήσης
Instrucciones de uso | Käyttöohje | Instructions d'utilisation | Upute za uporabu | Használati utasítás | Istruzioni per l'uso
使用説明 | 사용 지침 | Gebruiksaanwijzing | Bruksanvisning | Instrukcja użytkowania | Instruções de uso | Инструкция
по использованию | Návod na použitie | Navodila za uporabo | Uputstvo za upotrebu | Bruksanvisning | 使用说明

3211 Post-Op Bra, Bianca | 3212 Post-Op Bra, Bella | 3270 Post-Op Bra, Anna | 3291 Post-Op Bra, Alice | 3342 Post-Op Bra, Sophia | 3343 Post-Op Bra, Mary | 3373 Post-Op Bra, Marianne | 3444 Post-Op Bra, Ava | 3471 Post-Op Bra, Lisa | 3609 Post-Op Bra, Bonnie | 3831 Post-Op Bra, Bree | 3086 Comfort Bra, Catia | 5006 Comfort Bra, Celia | 5007 Comfort Bra, Celia

EN	4
AR	6
CS	8
DA	10
DE	12
EL	14
ES	16
FI	18
FR	20
HR	22
HU	24
IT	26
JA	28
KO	30
NL	32
NO	34
PL	36
PT	38
RU	40
SK	42
SL	44
SR	46
SV	48
CN	50
T 1-3 & 5	52-57

Intended purpose (Post-Op Bras): Fixate and support the breasts and protect wound and scar following breast surgery. Fixate bandages or other medical devices following breast surgery.

Intended purpose (Comfort Bras): Fixate and support the breasts and protect wound and scar when full movement of arm is regained following breast surgery and during/after radiation therapy.

Intended users: Women, following breast surgery as described under Indications.

Contraindications

- Relative contraindications:
- Very sensitive scar and skin tissue in the operated area
 - Skin diseases
 - Radiodermatitis
 - Contact dermatitis

Indications

For 24 hour post-surgical use, following the surgeries in ① 1

Clinical benefits

- Fixation of bandages and wound dressings
- Protection of wound/scar
- Support and compression of the operated breast tissue

Compression Level

The Post-Surgical Bras are ranked according to each other in five levels of compression as experienced based on the compression and support zones. High (1), Firm (2), Intermediate (3), Moderate (4) and Light (5). Figure shows a compression level of 2 (Firm) and is displayed on the packaging for the specific bra and in ① 2

Compression and support zones (Tytex ZoneTech®)

Table 4 – Firmness indicates the different knitted zones within the specific Post-Surgical Bra. The lower the number, the firmer the knitting; increased compression and support. Compression and support zones for each specific bra are displayed in ① 3

ZONE	FIRMNESS
1	1
2	2
3	3
4	4
5	5
6	6

① 4 – Firmness

- The knitting is unique to each specific Post-Surgical Bra and cannot be compared across the bra range.

Care instructions

- Find care instructions on the bra
- Close the bra before washing
- Wash with similar colours
- Wash in a laundry bag
- Use of chlorine based products may decrease durability and function

Reporting obligation

If you have any complaints, please contact your supplier directly. In the events of a serious incident that could lead to a significant deterioration of health or to death, contact the manufacturer and the competent authority of the member state. Serious incidents are defined in article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Lifetime of the device in use

The lifetime of the medical device is determined by natural wear and tear, even when treated correctly and as recommended. In case of signs of deterioration affecting performance and comfort, the device should be removed.

Warnings and precautions

- Follow the Instructions for Use
- Follow your surgeon's recommendations regarding use and fitting
- If any unexpected discomfort or wound irritation occurs, remove the Post-Surgical Bra and contact your surgeon
- The Post-Surgical Bra is not intended for direct contact with injured skin or mucous membrane
- Make sure that bandages, textile breast prosthesis and/or drainage tubes are placed correctly to avoid pressure points and/or obstructed flow
- Before radiotherapy and X-ray examination remove Post-Surgical Bra except; 3211 Post-Op Bra, Bianca, 3086 Comfort Bra, Calia and 5006/5007 Comfort Bra, Celia
- Wearing an undersized Post-Surgical Bra may cause local pressure pain and disruption of blood flow and/or lymphatic flow
- Wearing an oversized Post-Surgical Bra may reduce the clinical effect

⚠ Use the Post-Surgical Bra according to medical indication to avoid injury or damage.

Performance characteristics

General performance characteristics

- Compression and support zones: Integrated and targeted compression and support zones.
- 4-way stretch: The bra adjusts to the operated breast along with the patient's changing breast contour after surgery.
- Seamless: No seams in critical areas (cups, sides, back and under bust).
- Flexible sizing: Bra fits several sizes due to 4-way stretch and size concept.
- Breathable: Knitted material is breathable.

Specific performance characteristics ① 2

Material composition

Raw materials and components used in the Post-Op Bras are tested for harmful substances according to STANDARD100 by OEKO-TEX®, product class II.

The Post-Op Bras and packaging are:

- Biologically safe, non-toxic; chlorine and dioxin free, and do not contain human or animal tissue or extracts, prohibited azo-dye pigments, medical substances or phthalates
- Free from and unexposed to natural rubber latex or natural rubber derivatives
- Free from heavy metals or other harmful contamination and do not pose any harmful human impact associated with its intended end-use
- In compliance with REACH
- No chemicals added during treatment/finishing of the device. The device may contain softening and antistatic agents up to 1.5 %

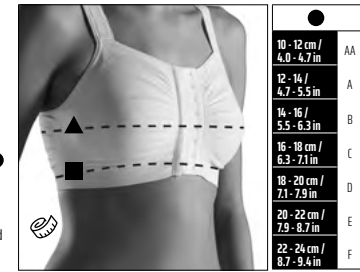
① 5

PRODUCT	MAIN FABRIC COMPOSITION
3211 Post-Op Bra, Bianca	77% CO, 13% PA, 10% EL
3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3270 Post-Op Bra, Anna 3086 Comfort Bra, Calia	91% PA, 9% EL
3342 Post-Op Bra, Sophia	78% CO, 15% PA, 7% EL
3343 Post-Op Bra, Mary	5-2XL: 91% PA, 9% EL 3XL-4XL: 94% PA, 6% EL
3831 Post-Op Bra, Bree 3444 Post-Op Bra, Ava	89% PA, 11% EL
3471 Post-Op Bra, Lisa	55% PA, 33% CO, 9% EL, 3% PES
5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	60% CO, 25% PA, 15% EL

Measuring bra size

1. Use a standard measuring tape
2. Stand up while measuring
3. Bust size ▲: Measure around the upper body, at the fullest part of the breast
4. Band size ■: Measure around the upper body, just under your breast
5. The difference between the bust size (▲) and the band size (■) shows your cup size (●) (▲ ■ = ●)
6. Find your bra size* in the size chart printed on the packaging** by using the band size ■ and the cup size ●

- ⚠ * Due to possible post surgical swelling and edema, you may need to go 1-2 sizes up compared to your normal bra size. Consult your surgeon for guidance.
- ⚠ ** The size guide printed on the packaging is unique to each specific Post-Surgical Bra and cannot be used across the bra range.

Instructions for use

Post-Surgical Bras in primary packaging are prepared for direct clinical use.

- Depending on medical indication, Post-Surgical Bras can be combined with 8113 Implant Stabilizer Band + 8108 Breast Puff can be used with pocketed Post-Surgical Bras

Fitting the bra and accessories

Post-Op Bras

STEPS	PRODUCT(S)	INSTRUCTION
1.	All Post-Op Bras	If medical condition allows, insert one arm and pull the bra over the shoulder. Then insert the other arm. If necessary, ask clinical staff for assistance.
2a.	3373 Post-Op Bra, Marianne 3609 Post-Op Bra, Bonnie 3212 Post-Op Bra, Bella 3291 Post-Op Bra, Alice 3343 Post-Op Bra, Mary 3831 Post-Op Bra, Bree	Pull the under bust band together. Fasten the bottom hook first and work up to the top. Use the widest setting immediately following surgery and the narrowest setting when swelling has subsided.
2b.	3211 Post-Op Bra, Bianca	Pull the under bust band together. Work from the bottom to the top, pressing firmly on hook & loop to close.
2c.	3342 Post-Op Bra, Sophia 3471 Post-Op Bra, Lisa	Pull the middle of the bra together. Fasten the middle hook first, then the other hooks in random order, followed by the zipper.
2d.	3444 Post-Op Bra, Ava 3270 Post-Op Bra, Anna	Pull the bra together. Fasten the top hook first, then the bottom, followed by the zipper.
3.	All Post-Op Bras	Position each breast correctly in the cup. Avoid pulling the bra down too low.
4.	All Post-Op Bras	If the bra has adjustable shoulder straps, adjust these so the bra; does not irritate the skin, provides support under the breasts and sides and feels comfortable to wear.

Comfort Bras

STEPS	PRODUCT(S)	INSTRUCTION
1.	3086 Comfort Bra, Calia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Pull bra over your head or pull it up like underwear.
2.	3086 Comfort Bra, Calia 5006 Comfort Bra, Celia 5007 Comfort Bra, Celia	Adjust the bra and ensure nothing is folded.

Textile Breast Prosthesis

STEPS	PRODUCT(S)	INSTRUCTION
	3373 Post-Op Bra, Marianne 3342 Post-Op Bra, Sophia 3343 Post-Op Bra, Mary 3471 Post-Op Bra, Lisa 3444 Post-Op Bra, Ava	Remove fiberfill from the device until size fits.
		Place the device in the pocket before bra is fitted.

Drainage Bulb

STEPS	PRODUCT(S)	INSTRUCTION
	3373 Post-Op Bra, Marianne	Place drainage bulb in loop and place loop on bra. Press firmly on hook & loop. Max 500 ml.

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

3211 Post-Op Bra, Bianca	3	-	-	-	-	-	-	-
3373 Post-Op Bra, Marianne	2	-	✓	-	-	✓	✓	✓
3609 Post-Op Bra, Bonnie	3	-	✓	-	-	✓	✓	✓
3212 Post-Op Bra, Bella	3	-	✓	-	-	-	-	-
3291 Post-Op Bra, Alice	4	-	✓	-	-	✓	✓	✓
3270 Post-Op Bra, Anna	4	✓	-	-	-	-	-	-
3342 Post-Op Bra, Sophia	1	✓	-	-	-	✓	✓	✓
3343 Post-Op Bra, Mary	2	-	✓	-	-	✓	✓	✓
3831 Post-Op Bra, Bree	2	-	✓	-	-	✓	✓	✓
3444 Post-Op Bra, Ava	2	✓	✓	-	-	✓	✓	✓
3471 Post-Op Bra, Lisa	1	✓	✓	-	-	✓	✓	✓
3086 Comfort Bra, Catia	4	-	-	-	-	-	-	-
5006 Comfort Bra, Tella	5	-	-	-	-	-	-	-
5007 Comfort Bra, Tella	5	-	-	-	-	-	-	-

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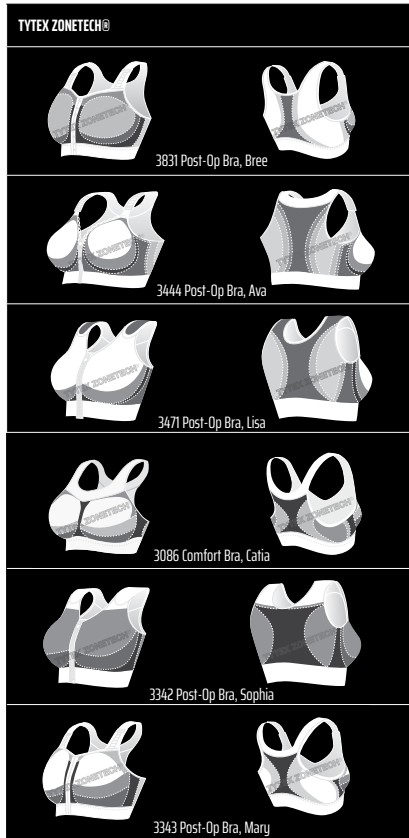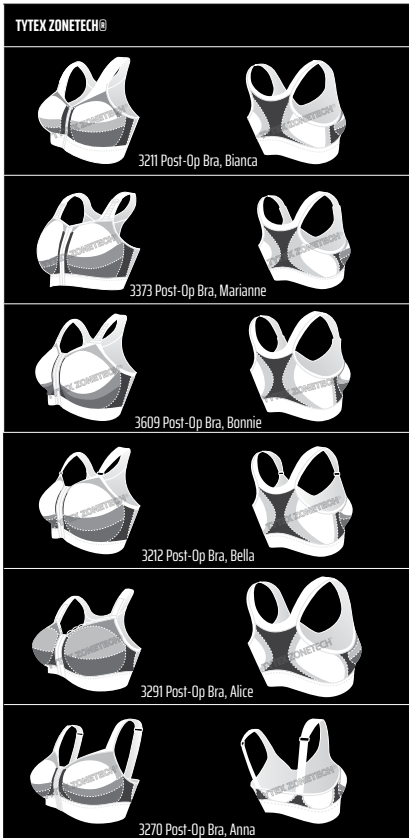carefix[®]

Carefix is a registered trademark of Tytex A/S © 2023.
All rights reserved Tytex A/S, Marsvej 6, DK-7430 Ikast

 Tytex A/S has been certified according to ISO 9001, ISO 14001 and ISO 13485
TYTEX A/S • Marsvej 6 • DK-7430 Ikast • www.tytex.com

FOLLOW TYTEX